

WECK HORIZON LIGATIONSYSTEM

The name you know the product you trust





THE CARTRIDGE

EASY LOADS WITH ADHESIVE CONVENIENCE

- The Horizon clip cartridge can provide incredibly easy and precise loading that can enhance surgical procedure efficiency.
- Lateral clip-restraining springs in the Horizon cartridge are displaced by applier jaws during clip loading.
- Spring displacement frees the clip for a nearly frictionless one-hand load from the cartridge.
- Clips are centered in the cartridge slots for precise loads.
- Clip cartridges are colour-coded to match appliers.
- Both 6- and 24-clip cartridges are available.
- A special adhesive backing on the Horizon cartridge allows the user to conveniently relocate the site of clip loading – cartridges affix to any sterile area.



THE APPLIERS

PERFORMANCE AND CLEANING ACCESSIBILITY

- The Horizon endoscopic applier provides clip closure designed to give surgeons solid tactile feedback.
- The large, lightweight rotation knob allows for one-finger, 360° rotation of the applier shaft.
- Horizon endoscopic appliers are available for medium, medium-large, and large Horizon clips, and feature flush ports for thorough cleaning.
- The reusable appliers are designed specifically for use with Horizon ligating clips, creating a dramatic cost savings when compared to disposable automatic-feed clip appliers.
- All Horizon open surgical ring-handled appliers open at the box lock for easy, thorough cleaning.
- Open surgical appliers are available with angled, curved, or right-angled jaws.

THE HORIZON SYSTEM: DESIGNED TO MEET YOUR NEEDS

Horizon is designed to meet the needs of surgeons, nurses, and other operating room personnel.



THE CLIP

FUNCTIONAL EXCELLENCE

- The chevron shape of the Horizon clip engulfs tissue with precise tip-to-tip closure.
- The heart-shaped wire is designed to give each clip a firm grip on vessels.
- Transverse grooves make the clip resistant to slipping across the vessel.
- The internal groove of the Horizon clip is sized to reduce clip scissoring.
- The triangulated cross-section of the Horizon clip leg maximises surface-to-surface contact of clip and applier jaw, practically eliminating clip fallout.

☐ Micro
 ☐ Small
 ☐ Small/Wide
 ☐ Medium
 ☐ Medium/Large
 ☐ Large

LIGATING CLIPS

TITANIUM LIGATING CLIPS

WECK

REF.	CLIP SIZE	CLIPS/CARTRIDGE	CARTRIDGES/BOX	CLIPS/BOX
005200	○ microclip	6	30	180
001200	● small	6	30	180
001204	● small	24	25	600
001201	● small-wide	6	30	180
001205	● small-wide	24	25	600
002200	● medium	6	30	180
002204	● medium	24	25	600
003200	● medium-large	6	20	120
003204	● medium-large	24	15	360
004200	● large	6	20	120
004204	● large	24	15	360

OPEN SURGERY

LIGATING APPLIERS

WECK

REF.	CLIP SIZE	WORKING LENGTH	QTY
20° curved			
537061	○ microclip	6" (15 cm)	1
537081	○ microclip	8" (20 cm)	1
137061	● small	6" (15 cm)	1
137081	● small	8" (20 cm)	1
137111	● small	11" (28 cm)	1
137062	● small-wide	6" (15 cm)	1
137082	● small-wide	8" (20 cm)	1
137112	● small-wide	11" (28 cm)	1
237061	● medium	6" (15 cm)	1
237081	● medium	8" (20 cm)	1
237111	● medium	11" (28 cm)	1
337081	● medium-large	8" (20 cm)	1
337111	● medium-large	11" (28 cm)	1
437081	● large	8" (20 cm)	1
437111	● large	11" (28 cm)	1
55° angled			
137085	● small	8" (20 cm)	1
137086	● small-wide	8" (20 cm)	1
237085	● medium	8" (20 cm)	1
237115	● medium	11" (28 cm)	1
337085	● medium-large	8" (20 cm)	1
337115	● medium-large	11" (28 cm)	1
437085	● large	8" (20 cm)	1
437115	● large	11" (28 cm)	1
70° angled			
237117	● medium	11" (28 cm)	1
337117	● medium-large	11" (28 cm)	1
437117	● large	11" (28 cm)	1

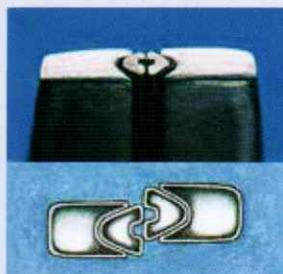
All Horizon™ ligating clips are packaged sterile. Ligating clip systems differ in their clip-closure characteristics. It is the responsibility of the user to confirm security of clips on vessels or other tissue. Not intended for contraceptive tubal occlusion.

MICROCLIP™

Horizon's newest family member. For safe ligation of small vessels or structures.

SPECIALTY MULTI-ANGLE APPLIERS

The Horizon 28 cm Multi-Angle Access Applicator for small (wide slot) clip cartridges incorporates a 45° angled jaw, or a curved jaw, for an optimal angle of access to the surgical target.

CROSS-SECTIONAL VIEW OF CLIP

With applicator jaws properly aligned (above) and misaligned (below).

ENDOSCOPIC TAKE-APART CLIP APPLIER

Our TAP applicators can be taken apart for easy cleaning. They come with a 360° rotation knob for better access and improved visibility.

OPEN SURGERY**LIGATING APPLIERS, SPECIAL****WECK**

REF.	CLIP SIZE	APPLIER DESIGN	WORKING LENGTH	QTY
20° curved				
137118	● small-wide	multi-angled	11" (28 cm)	1
45° angled				
137117	● small-wide	multi-angled	11" (28 cm)	1

ENDOSCOPIC SURGERY**TAKE-APART LIGATING APPLIERS****WECK**

REF.	CLIP SIZE	Ø APPLIER	Ø TROCAR	WORKING LENGTH	QTY
238110T	● medium	10 mm	10 mm	13" (33 cm)	1
338110T	● medium-large	10 mm	10 mm	13" (33 cm)	1

NON-TAKE-APART LIGATING APPLIERS**WECK**

REF.	CLIP SIZE	Ø APPLIER	Ø TROCAR	WORKING LENGTH	QTY
438110	● large	12 mm	12 mm	11" (28 cm)	1

NON-TAKE-APART LIGATING APPLIERS, ENDOSAPHENOUS**WECK**

REF.	CLIP SIZE	Ø APPLIER	Ø TROCAR	WORKING LENGTH	QTY
238200	● medium	5 mm shaft, 7 mm jaw	8 mm	13" (32 cm)	1

Teleflex is a leading global provider of specialty medical devices used for diagnostic and therapeutic procedures in critical care, urology and surgery. Our mission is to provide solutions that enable healthcare providers to improve outcomes and enhance patient and provider safety. We specialise in devices for general and regional anaesthesia, cardiac care, respiratory care, urology, vascular access and surgery and we serve healthcare providers in more than 150 countries. Teleflex also provides specialty products for medical device manufacturers.

Our well known brands include ARROW®, DEKNATEL®, GIBECK®, HUDSON RCI®, KMEDIC®, LMA™, PILLING®, PLEUR-EVAC®, RÜSCH®, SHERIDAN®, TAUT®, TFX OEM®, VASONOVATM, VIDACARE® and WECK®, all of which are trademarks or registered trademarks of Teleflex Incorporated.

Teleflex global operations: Australia, Austria, Belgium, Canada, China, Czech Republic, France, Germany, Greece, India, Ireland, Italy, Japan, Malaysia, Mexico, Netherlands, New Zealand, Portugal, Singapore, Slovak Republic, South Africa, Spain, Switzerland, United Kingdom, Uruguay and USA.

YOUR INTERNATIONAL CONTACTS:

TELEFLEX HEADQUARTERS INTERNATIONAL, IRELAND

Teleflex Medical Europe Ltd., IDA Business and Technology Park,
Dublin Road, Athlone, Co Westmeath
Phone +353 (0)9 06 46 08 00 · Fax +353 (0)14 37 07 73
orders.intl@teleflex.com

AUSTRALIA/NEW ZEALAND +61 (0)3 9081 0600

AUSTRIA +43 (0)1 402 47 72

BELGIUM +32 (0)2 333 24 60

CHINA (SHANGHAI) +86 (0)21 6381 6399

CHINA (BEIJING) +86 (0)10 6418 5699

CZECH REPUBLIC +420 (0)495 759 111

FRANCE +33 (0)5 62 18 79 40

GERMANY +49 (0)7151 406 0

GREECE +30 210 67 77 717

INDIA +91 (0)44-2836 5040

ITALY +39 0362 58 911

JAPAN +81 (0)3 3379 1511

NETHERLANDS +31 (0)88 00 215 00

PORTUGAL +351 22 541 90 85

SINGAPORE +65 6439 3000

SLOVAK REPUBLIC +421 (0)3377 254 28

SOUTH AFRICA +27 (0)11 807 4887

SPAIN +34 918 300 451

SWITZERLAND +41 (0)31 818 40 90

UNITED KINGDOM +44 (0)1494 53 27 61

For detailed information see www.teleflex.com

The products in this catalogue may not be available in all countries.
Please contact your local representative. All data current at time of printing (03/2014).
Subject to technical changes without further notice.

94 05 37 - AP 00 01 - REV A - MC / PDF - 06 13 PDF